



The Ferno
Mk 1 and Mk 4 Mobylye Carry
Chairs

User Manual

1. Intended Use of the Mobylye Carry Chair

The Ferno Mobylye Carry Chair is used by various Commercial, Military Ambulance and Welfare Services throughout the World.

The Mobylye Carry Chair is intended for use by fully qualified, trained and competent carers, attendants, paramedics or other such medical staff as an aid to provide safe and secure transportation of patients.

2. Mobylye Carry Chair Features

Thank you for purchasing your Ferno *Mobylye Carry Chair*.

The design of the Mobylye Carry Chair is to aid the movement of a patient in a seated position by transporting in situations where a larger device, such as a trolley or stretcher, cannot be used or manoeuvred.

The Mobylye Carry Chair is also equipped with one flexible restraint for patient comfort and security.

The Mobylye Carry Chair is also designed to fold, in a compact and flat position, for shipping and storage purposes. If stored within an ambulance, ensure that it is secured in a safe and unobtrusive position.

The Mobylye Carry Chair is designed to be available with 5" wheels and is manufactured in mild steel with a painted finish.

WARNING

The Mobylye Carry Chair should only be used by fully trained personnel.

The installation of a Mobylye Carry Chair within an ambulance or vehicle, plus the supply of the correct fixings, is the responsibility of the installation engineer.

This product must be regularly serviced and maintained to ensure satisfactory operation.

Maintenance of the Mobylye Carry Chair is to be carried out by fully trained personnel only.

SAFE WORKING LOAD

114Kg

WARNING

The Mobylye Carry Chair is to be used by fully trained personnel only. Trapping hazards cannot be eliminated due to the operation of this product. EXTRA CARE to be taken whilst folding and unfolding.

ESSENTIAL REQUIREMENT

Every three months lubricate the hinge pins with light oil paying particular attention to the cross section pins.

3. Checking the Mobylye Carry Chair

The following checks are required each day before the use of the Mobylye Carry Chair:

1. Overall cleanliness of the Mobylye Carry Chair.
2. Check that there are no bent or damaged parts, missing fasteners, etc.
3. Check that the wheels rotate freely with no excess movement or damage.
4. Check that the Mobylye Carry Chair unfolds and locks securely in the seating position for normal use.
5. Check the upholstery for cracks, splits or any loose fabric or stitching, etc.
6. Check that the upholstery is correctly secured to the frame.
7. Check that the patient restraint is in good condition.

If in any doubt then contact a Service Engineer.

<p>Regular cleaning helps reduce the risk of transmitting disease and enables the equipment to function at its optimum.</p>
--

4. Operating Procedure

The Mobylye Carry Chair is normally in the folded position when either not in use or in storage. To unfold the Mobylye Carry Chair proceed as follows:

1. Position the Chair vertically by holding the Chair back top frame with one hand. With the other hand, grip the side of the seat frame and pull upwards. This will position the wheels on the floor.
2. Still holding the Chair back frame with one hand, adjust the seat with the other hand until the seat frame rear crossbar drops securely into the back frame locking bar, ensuring that the seat frame is seated correctly and safely.
3. The Mobylye Carry Chair is now ready for use. To fold the Mobylye Carry Chair, reverse the above procedure.

General Principles

- Use a recognised patient handling technique to place a patient on the Mobylye Carry Chair.
- After placement on the chair, secure the patient using the Mobylye Carry Chair chest restraint and never leave the patient unattended or unsupported, as the chair is designed to tilt back easily to aid handling. These procedures are essential for safety whenever a patient is seated on the chair.
- When moving patients using the Chair, always inform them of what you are going to do. Always follow good lifting procedures.
- With the patient in position on the Chair, the rear attendant grips the chair back frame, then tilts the patient / chair back until the weight is balanced evenly on the chair wheels. The Mobylye Carry Chair can now be rolled, without lifting.
- If the patient/chair require carrying, the same tilt back and balance procedure is used, then the attendants grip the front and rear carrying handles and lift simultaneously.

Additional Features – Mk 4 Only

The Mk 4 Mobylye Carry Chair has the following additional features: -

1. Foot bar facility
2. 4 wheels
3. Higher back frame

Important

In cases of cervical, spinal and some fracture injuries, use of the Mobylye Carry Chair for patient movement is not recommended.

5. Care of the Mobylye Carry Chair

Routine Cleaning

Always wash hands before and after cleaning the Mobylye Carry Chair. The use of protective clothing is recommended when equipment handled is known, or suspected, to be infected. Thorough cleaning is required where the Chair is dirty; wipe with detergent and warm water applied with a clean cloth. Dry with paper towels.

Infected equipment must be considered on an individual basis depending on the type of infection present. All equipment should be disinfected prior to re-use for another patient.

If the Mobylye Carry Chair was used in a known, or suspected, infectious environment, the disinfection procedure must be followed after the cleaning procedure.

Disinfecting

Disinfection procedure for the removal of most bacteria and viruses.

Whilst wearing appropriate protective equipment, prepare a suitable bleach solution in accordance with the manufacturer's instructions. If in any doubt as to the suitability of any solution, test it on a small area first.

Wipe Chair surfaces with the solution and dry with clean paper towels.

Once dry, preventative maintenance or routine servicing may be conducted.

Discard the protective equipment in an appropriate receptacle and wash hands.

Routine Maintenance

Periodically inspect the Mobylye Carry Chair for general deterioration, e.g. excess play in linkages, loose bolts, worn restraints or covers, etc.

Light lubrication on linkages and occasional waxing or polishing are all that is normally required to ensure trouble-free use.

6. Warranty and Service

Ferno's standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warrant terms and do not limit the statutory rights of the customer.

For service, maintenance and any questions regarding this, or any other Ferno product, please contact either our Internal Sales Department or Fieldcare Department:-

Ferno (UK) Limited
Ferno House
Stubs Beck Lane
Cleckheaton
West Yorkshire
England
BD19 4TZ

Telephone: +44 (0) 1274 851999

Fax: +44 (0) 1274 851111

or your local distributor.

Date	Service By	Remarks

7. Technical Data

Product Name:	MK 1 MOBYLE CARRY CHAIR	MK 4 MOBYLE CARRY CHAIR
Seat Height:	500mm	500mm
Seat Size:	340mm wide x 300mm deep	340mm wide x 300mm deep
Overall length folded:	1170mm	1250mm
Overall width folded:	410mm	410mm
Overall depth folded:	(5" Wheels) 220mm	(5" Wheels) 220mm
Weight:	(5" Wheels) 8.5kg	(5" Wheels) 10.1kg
Load Capacity:	114kg	114kg

Construction Materials:

- 1. Frame:** Mild steel
- 2. Finish:** Painted
- 3. Upholstery:** PVC



The CE mark on this product demonstrates conformity with the requirements of the EC Directive 93/42/EEC for Medical Devices.

Registered Office:

Ferno (UK) Limited
Ferno House, Stubs Beck Lane, Cleckheaton, West Yorkshire, England. BD19 4TZ
Tel: + 44 (0) 1274 851999 Fax: + 44 (0) 1274 851111
e-mail: service.uk@ferno.com

Registered number: 1007475 England

As our policy is one of continuous development Ferno (UK) Ltd reserves the right to change the specifications without notice.

Stock Code: 2003-0012