

Product specification

Product	CEDERROTH EYE WASH
Reference code	7251

1. General description A personal eyewash equipment containing sterile, buffered, saline eye wash solution. Stored in an easy opened, one-step activated bottle with an integrated eye cup and dust cover.
The dust cover also functions as an opening device.

2. Intended use The Cederroth Eye Wash is intended for washing away foreign particles and dangerous liquids, as alkali or acids, from an eye in an emergency first aid situation at work places.

The Cederroth Eye Wash is intended for the first acute phase at the very accident site before getting to a plumbed or self-contained eye wash equipment or seeking medical attention.

3. Composition

-Liquid:	Sterile, isotonic Sodium Chloride 0.9%, Buffered with Sodium Borate to neutral pH. Preservative free.	
-Volume:	500 ml	
-Bottle:	Leak proof polypropylene bottle produced with Blow-Fill-Seal Technique. Eye cup in semi-transparent polypropylene Dustcover in green hard plastics (ABS)	
-Bottle dimensions:	Diameter:	66 mm
	Height:	235 mm

4. Labelling

On bottle: Cederroth address,
Telephone- and faxnumber, e-mail address
Product name
Reference code
Lot number
Expiration date
CE-mark with notified body's number
Sterile
Only use once
Directions for use
EAN code

On Transport packing: Cederroth, address, telephone- and faxnumber
Name of the product
Reference code
LOT-number
Expiration Date
EAN-code
Number of products per package

5. Storage conditions

+2°C to +40°C. Do not freeze.
Store in the original packaging, for a maximum protection until time of use.

6. Accessories

Cederroth Wall Bracket for Eye Wash (REF 7200) for organized storage and defined placement. Also works as an anti-theft system.

7. Authorisation

Authorised by:

Yvonne Törnevik
Quality Assurance Manager

Date: 2002-07-05
